

Surgical technique of the Noviomagus Mini Mill



Application: This device is solely meant for milling cortico-cancellous bone into controlled size bone particles.

Instructions for assembly

1. Place housing on table surface.
2. Position clamp on base plate and push clamp's black spacers firmly against table edge while tightening screw spindle.
3. Insert milling drum with appropriate roughness into housing.
4. Position handle on hexagonal shaft of milling drum.
5. Tighten screw nut on milling drum's shaft end.
6. Insert push block into funnel.

Surgical technique

1. Choose milling drum with appropriate roughness.
2. Assemble Mini Mill according to assembly instructions.
3. Place bowl under milling drum outlet.
4. Fill funnel with bone graft and insert push block.
5. Rotate handle clockwise while pushing push block downwards.
6. Do not reinsert or exchange milling drum unless all grafts are removed from housing interior.


Safety instructions

1. Do not operate Mini Mill without push block into position.
2. Be careful for sharp teeth on milling drum when handling milling drum outside Mini Mill.



- For instructions for use see document "Instructions for use of reusable surgical instrumentation" (IFU-MOI001).
- For reprocessing of reusable medical devices see document "Reprocessing of reusable medical devices" (REP-MOI005).

LET OP: Onderkant afsnijden!
breedte x hoogte 21,0x24,0 cm

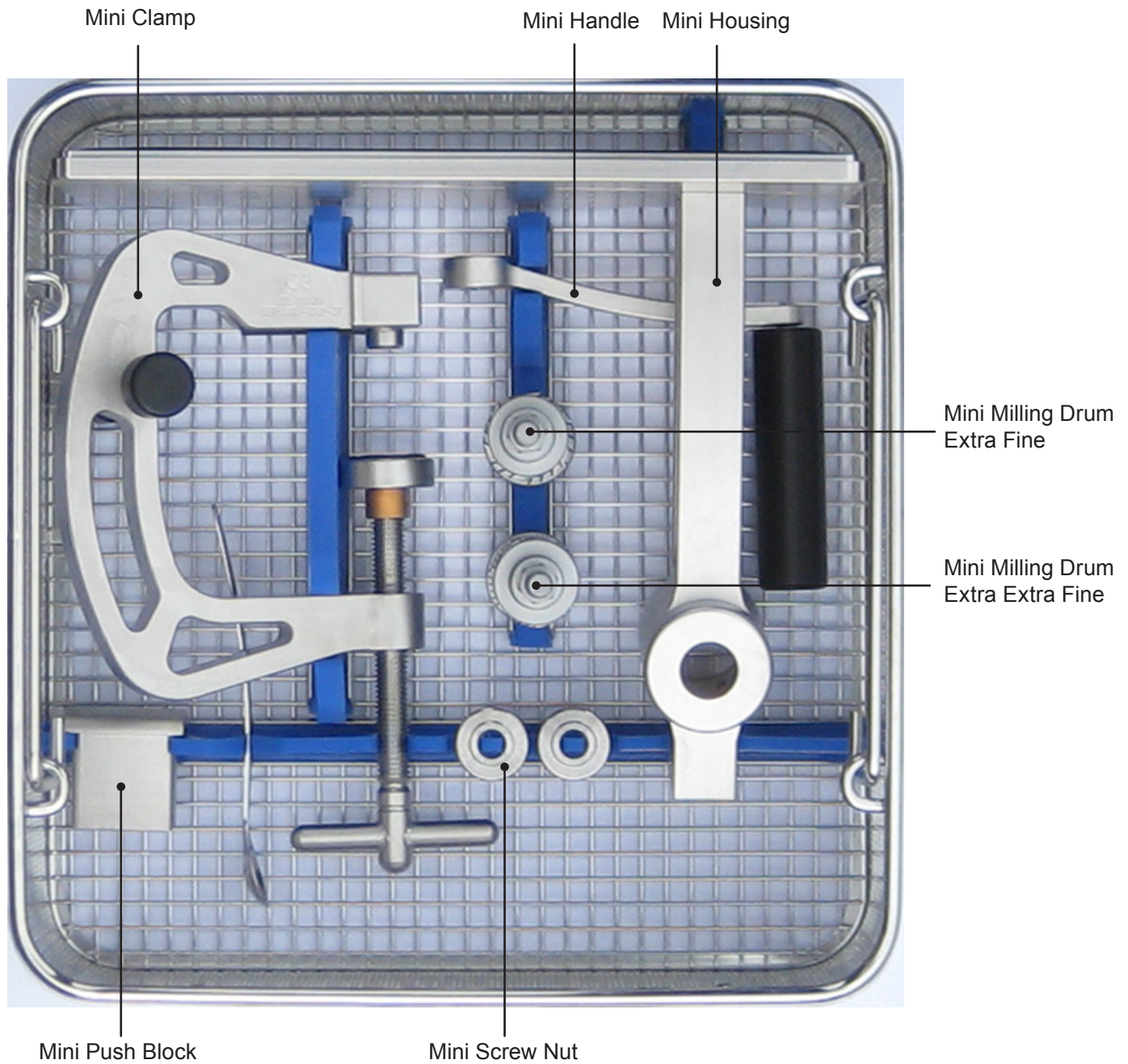

Spierings Orthopaedics B.V.

Madoerastraat 24

6524 LH Nijmegen

The Netherlands

Tel: +31 - 24 - 35 01 603

Fax: +31 - 24 - 35 01 604

©2012 Spierings Orthopaedics B.V. All rights reserved. Printed in the Netherlands.

February 2012, SUR-SBM001-3 EN

LET OP: Onderkant afsnijden!
 breedte x hoogte 21,0x24,0 cm